

The easy way to pull on socks or stockings.

Colour	White
Dimensions (mm)	235 (L)
Weight (kg)	0.048



1. Introduction

Thank you for deciding to purchase the Sock & Stocking Aid from Aidapt. The Sock & Stocking Aid is manufactured from the finest materials and components available. When used correctly it is designed to give many years of reliable trouble-free service.

2. Before Use

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the Sock & Stocking Aid.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

3. Instructions for Use

Using the Sock & Stocking Aid is simple, but requires a certain amount of dexterity:

- Place the sock or stocking over the pliable Sock & Stocking Aid.
- Place the foot into the opening of the sock/stocking, moving the foot down to the bottom of the sock/stocking.
- Using the two cotton straps, pull the Sock & Stocking Aid up the leg, this will pull the sock or stocking up at the same time.
- Carefully remove the Sock & Stocking Aid when the sock/stocking is positioned correctly.

4. Cleaning

Clean your Sock & Stocking Aid using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

5. Reissue

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/ bolts/castors
- Other screw in/ bolt in/ push in components.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

6. Important Information

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited or Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

7. Care & Maintenance

Please make a safety check of the product at regular intervals or if you have any concerns.